innoliving
easy tech, easy life

ALCOOL TESTER

INN-993
MANUALE D'USO

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ALCOHOL TESTER

**INN-993
USER MANUAL**

Thank you for purchasing the INNOLIVING alcohol tester, which uses a semiconductor gas sensor to detect the concentration of alcohol in the exhaled air and converts this measurement into a readable value that expresses the estimated concentration of alcohol in the blood.

BEFORE USE, READ THIS USER MANUAL CAREFULLY AND KEEP IT FOR FUTURE REFERENCE.

WARNINGS

- The tester provides you with information to determine your blood alcohol level at the same time you use the test. Blood alcohol concentration changes as time passes and measurement may be affected by environmental conditions. This product is designed to be used as reference

only in order to promote a responsible and safe driving. It provides a purely indicative measurement, it doesn't replace or exclude possible tests conducted by competent authorities, which can have different readings. The tester readings have no legal value and do not relieve any individual responsibility. Improper storage of the product and/or improper use of the product may affect the accuracy of the measurement. You should not rely upon it as the sole basis to determine state of drunkenness or whether it is safe to drive a vehicle, operate equipment, or engage in dangerous activities. Innoliving Spa is

not liable for the improper use of this product for any reason.

- Before use, make sure that the appliance is intact without visible damage. If in doubt, do not use the product and contact the authorized after sales service
- After use, always store the product away from sources of heat, sunlight, humidity, sharp objects or similar.
- Do not use alcoholic substances such as paint or insecticides near the tester.
- Do not use chemicals to clean the tester. Never immerse the product in water or other liquids, the sensor could be damaged.

- Do not attempt to repair the product yourself, under penalty of invalidation of the warranty, but always contact the authorized after sales service.
- It takes about 20 minutes for alcohol to be absorbed into the bloodstream by the digestive system; therefore, it is recommended to take the measurement 20 minutes after consuming alcoholic beverages. If the measurement is taken immediately after drinking, the results may reflect the concentration of alcohol in the drink and not the actual concentration in the blood
- To obtain more accurate measurement

results, the interval between two consecutive measurements should be more than 3 minutes.

PRODUCT DESCRIPTION

1. Air intake
2. LED indicator
3. LCD Display
4. ON/OFF button
5. Buzzer hole
6. Batteries room
7. Emergency LED light

HOW TO USE THE ALCOHOL TESTER

1. The device works powered by 3X 1.5V (AAA) batteries, not supplied. Remove the battery compartment cover, install the batteries correctly, and replace the battery compartment cover. In case of prolonged non-use, it is advisable to remove the batteries from the device.
2. Press the power button and hold for 2 seconds. You will hear a "beep" when turning on.
3. During the "pre-heat" phase (1~10 seconds), the device starts a 10-second countdown for pre-heat. Wait for pre-heating to complete.
4. After pre-heating, the device automatically switches to the detection countdown, which lasts 6 seconds, Blow into the air vent for at least 3 seconds and in any case until the "beep" indicates to stop. Then wait for the test results to appear.
5. To change the displayed unit of measurement: After completing the operations described in points 2, 3 and 4, within 5 seconds of the result appearing on the display, press once to move to the next unit of measurement successiva (%BAC -, g/L, mg/L).
6. **Turning on the emergency light:** Press the power button twice while the device is turned off to turn on the emergency LED light. To turn it off, briefly press the same button while the light is on.

Important note:

When you turn on the device for the first time (for example, just unpacked or after a long period of non-use), residual gases or impurities may accumulate inside the sensor. In this case:

- Press the power button several times to start the pre-heat cycle and "clean" the device to ensure accurate measurements.

DISPLAY SYMBOLS AND LUMINOUS ALERTS

WAIT: after turning on the appliance, the word "WAIT" appears on the display to indicate the pre-heating phase.

BLOW: once pre-heating is complete, the display indicates "BLOW" to start blowing on the air vent.

RESULT: when the detected value is displayed, the writing "RESULT" appears on the display and the relative measurement value expressed in one of the selectable units of measurement (%BAC -, g/L, mg/L).

OFF: "OFF" appears when user turns the unit off or when the batteries are LOW, in this case, the unit emits also a low power alarm, reminding the user to replace the batteries.

Depending on the value detected, the light indicator takes on a different colour:

- Green: result from 0,0g/L to 0,29g/L
- Yellow: result from 0,3g/L to 0,49 g/L
- Red: result \geq 0,5g/L

RATING LABEL

Innoliving Spa
Via Merloni 2/B - 60131 Ancona-Italy
INN-993 Alcool Tester
4,5V --- 3xAAA batt.
MADE IN CHINA
Lot n.

The product complies with all the applicable European regulations.

INFORMATION TO USERS according to Legislative Decree No. 49 of March 14, 2014 " 2012/19/UE Implementation of the Directive on Waste Electrical and Electronic Equipment (WEEE) "

The crossed-out dustbin symbol indicates that the product at the end of its life must be collected separately from other waste. The user should, therefore, take the equipment with the essential components at the end of its useful life to the separate collection center of electronic and electrical waste, or return it to the retailer when purchasing new equipment of equivalent type, in ratio of one to one, or one to zero for devices with larger side less than 25 CM. The separate collection for the decommissioned equipment for recycling, treatment and environmentally compatible disposal contributes to avoid possible negative effects on the environment and human health and promotes recycling of the materials. Improper disposal of the product by the user entails the application of administrative sanctions according to Legislative Decree No. 49 of 14 March 2014

innoliving
easy tech, easy life

Innoliving Spa
Via Merloni, 2/B - 60131 Ancona Italy
Tel 071.2133550 - www.innoliving.it

MADE IN CHINA

Rev00_01_2025